Name of Policy:	Weight Limits of Cardiovascular Suite Fluoroscopy Tables	THE UNIVERSITY OF TOLEDO
Policy Number:	3364-102-02	MEDICAL CENTER
Department:	Cardiovascular Lab	
Approving Officer:	Chief Operating Officer - UTMC	
Responsible Agent:	Director, Cardiovascular Services	
Scope:	University of Toledo Medical Center	Effective Date: 03/2022 Initial Effective Date: 09/23/2011
New policy proposal Major revision of existing policy X Reaffirmation of existing policy		

## (A) Policy Statement

To ensure no patient is placed on the Fluoroscopy Tables that exceeds the manufactures weight limit for safe operation.

## (B) Purpose of Policy

To comply with the Safe Medical Device Act of 1990 and provide a safe procedural environment for all patients of the Cardiovascular/EP Lab.

## **Procedure**

The weight limit of the EP Lab Fluoroscopy Table is 204kg (448 lbs.) The weight limit of Suite 2, 3 Fluoroscopy Tables is each 220kg (484 lbs.).

- (1) Patient weight will be obtained from the patient's medical record. If patients weight is within 15 kg (30 lbs.) of the table limit:
  - (a) Patient will be weighted prior to scheduled procedure for a real time accurate weight.
  - (b) If patient found to be under the weight limit of the table the procedure will continue as scheduled.
  - (c) If patient is found to be over the weight limit the procedure physician will be notified and the procedure will be rescheduled or cancelled.

Approved by:		Review/Revision Date: 09/2011	
Todd Korzec, RN, BSN Director, Cardiovascular Services	03/18/2022 Date	07/2014 03/2017 03/2020 03/2022	
Ehab Eltahawy, MD, MPH Medical Director, Cardiovascular Lab	03/18/2022 Date		
/s/ Christine Stesney-Ridenour, FACHE Chief Operating Officer- UTMC	03/18/2022 Date		
Review/Revision Completed By: Todd Korzec		Next Review Date: 03/2025	
Policies Superseded by This Policy:			